

DONNING GUIDE

BioClean-C™ Apron BCDA/S-BCDA
Chemotherapy Protective Apron

- Undergarments, hairnet and cleanroom clogs/shoes are already donned before entering the changing room
- Movements should be slow and controlled
- Do not talk, cough or sneeze whilst in the donning area

1



Remove the apron from the packaging.

2



Hold the apron on the inner surface and unfold. Ensure apron is facing the correct way round with the smooth PE surface facing outwards (the garment label will be on the inside face of the apron).

3



Put the apron with the neck strap over the shoulders. Loop the neck strap behind the neck and connect both ends.



4



Hold the loop of the knotted tie in one hand and the loose end of the knotted tie in the other.



Take the loose end of the knotted tie and bring it to the loop, so you are holding both in the same hand.

5



Take the loose end of the unknotted tie and bring it to the hand with the knotted tie, then take the end of the knotted tie in your other hand and pull the ends (in the right hand) and the folded section (in the left hand) of the tie apart, the knot should undo easily.

6



Tie the waist strap at the side for a secure fit.

7



Your apron is now donned. Ensure all the coverall and accessories are correctly fastened and secured prior to entering the cleanroom.

This donning guide is for guidance only, site protocols and procedures should be followed at all times.



Need a detailed guide?
Scan the QR code to watch the donning video and ensure proper aseptic technique.
Visit www.ansell.com for more cleanroom safety solutions.

