



DIRECTIONS FOR USE: Correct Medication Labeling System

Pat. # US 6,955,002 B2; 7,381,288;
D538,851; D542,415


- 1 WRITE IN APPROPRIATE
INFORMATION USING A PERMANENT MARKER

Afrin Strength: _____ Expiration Date / Time: _____	Marcaine Strength: _____ % Expiration Date / Time: _____
Afrin Strength: _____ Expiration Date / Time: _____	Marcaine & Epinephrine Strength: <u>5 % / 1:200,000</u> Expiration Date / Time: <u>2-24</u> <u>12:00</u>
Antibiotic ± 0.9% Saline Irrigation Name and Strength: _____ Expiration Date / Time: _____	Papaverine Strength: _____ Expiration Date / Time: _____
Contrast Strength: _____ Expiration Date / Time: _____	Contrast Strength: _____ Expiration Date / Time: _____
Lidocaine Strength: <u>2 %</u> Expiration Date / Time: <u>2-24</u> <u>12:00</u>	Lidocaine Strength: <u>2 %</u> Expiration Date / Time: <u>2-24</u> <u>12:00</u>

- 2 ADHERE LABEL
ONTO SYRINGE

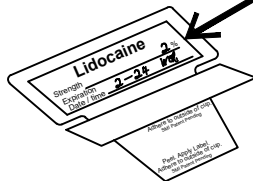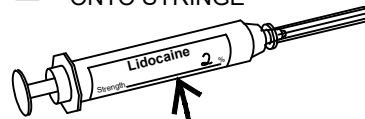


- 3 PEEL PAPER BACKING
HALFWAY BACK. ADHERE
LABEL TO FLAG SURFACE.
USE PRE-PRINTED OR
MARK ON LABELS.

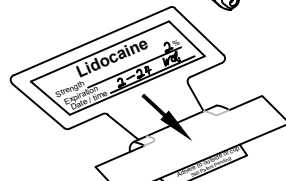

- 4 PEEL PAPER BACKING OFF
TO EXPOSE ADHESIVE.
COLORED SYRINGE LABELS
WILL COME OFF WITH BACKING.

- 6 PEEL OFF SYRINGE LABEL.
ADHERE CORRESPONDING
COLORED LABEL TO
SYRINGE BARREL.

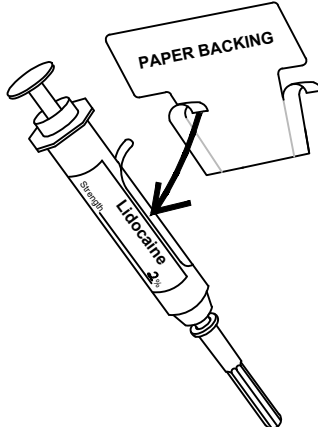


- 7 SET SPECIMEN STRIP AND
MARKING PEN ON BACK
TABLE.

1 SPECIMEN ZONE A	2 SPECIMEN ZONE B	3 SPECIMEN ZONE C	4 SPECIMEN ZONE D
LEFT RIGHT	LEFT RIGHT	LEFT RIGHT	LEFT RIGHT


TIME OUT.

REMEMBER TO TAKE A


TIME OUT.

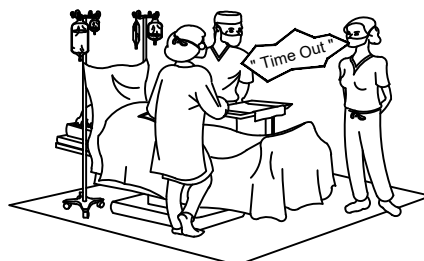

- 5 ADHERE FLAG TO OUTSIDE OF CUP.
OPTIONAL:
REMOVE ADHESIVE BACKING
FROM BOTTOM OF CUP AND
THEN ADHERE TO A SURFACE.

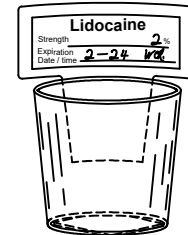

- 8 PLACE SPECIMEN ON
SQUARE. WRITE SPECIMEN
NAME DIRECTLY ON PAD.
USING PERMANENT INK.

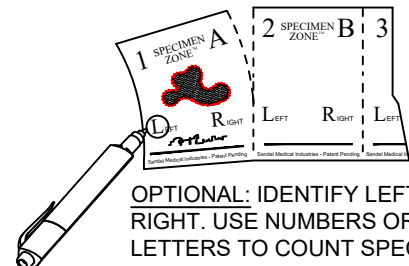

OPTIONAL: IDENTIFY LEFT OR
RIGHT. USE NUMBERS OR
LETTERS TO COUNT SPECIMENS.

STERILE R
Sterilized by Gamma Radiation



Do Not
Resterilize



Do Not Reuse



Latex

SANDEL LLC
Leaders in Healthcare Safety®

Ansell

Ansell Sandel Medical Solutions LLC
9301 Oakdale Avenue Suite 300,
Chatsworth, CA 91311
818-534-2500 www.sandelmedical.com