


# DIRECTIONS FOR USE: STRETCH-A-TRAY™

Reorder No. 1147

1. PLACE ON MAYO STAND, BACK TABLE STAND, BACK TABLE OR PATIENT.  
O P T I O N A L: PEEL ADHESIVE TAB AT BOTTOM OF TRAY AND ADHERE  
IN DESIRED POSITION.
2. ADJUST TRAY TO DESIRED LENGTH.
3. PLACE SHARP ITEMS IN TRAY WHEN PASSING.

CAUTION: ENSURE SHARP INSTRUMENTS ARE PROPERLY POSITIONED  
INSIDE THE TRAY TO AVOID POTENTIAL INJURIES.

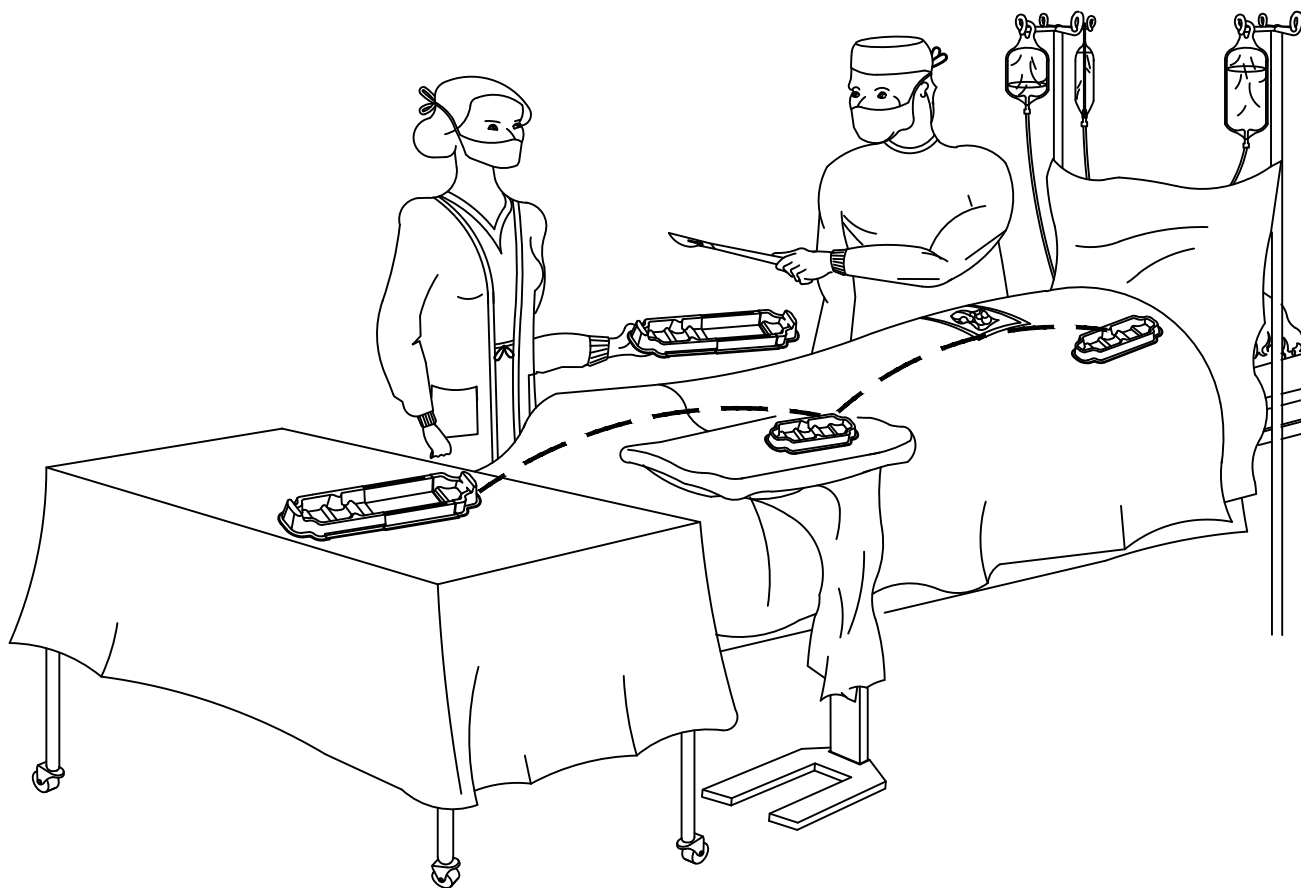

Do Not  
Resterilize



Single use Only



  
Leaders in Healthcare Safety

Ansell Sandel Medical Solutions, LLC  
9301 Oakdale Avenue Suite 300  
Chatsworth, CA 91311  
1-818-534-2500 [www.sandelmedical.com](http://www.sandelmedical.com)