

Step by Step Guide to Dressing an O.R. Table with STAT-BLOC™ Linens

STAT-BLOC™ disposable linens with AMT™ Antimicrobial Technology provide complete protection against bacterial migration and are at least 99% effective against MRSA, CRE, E. Coli and COVID-19.* Room Turnover Products include disposable linens, patient safety straps, mops, bags, solidifiers and turnover kits to reduce the risk of cross-contamination, improve turnover time, and improve patient outcomes.



1. After ensuring the O.R. table has been thoroughly cleaned and dry time followed per IFU of manufacturer, obtain assembled kit and open.



2. Unfold the table sheet lengthwise and tuck it around the bed, just as you would a traditional linen.



3. Unfold the draw sheet on both sides. The SANDEL™ draw sheet can hold up to 1000 pounds.



4. Unfold and place the STAT-BLOC™ arm board covers by simply sliding them onto the arm boards. Standard STAT-BLOC™ arm board covers feature two straps attached with loops.



5. Follow the hospital's policies and procedures to place the safety strap. Each SANDEL® safety strap features a velcro securement square to keep it off of the floor, preventing contamination.



6. O.R. table is now dressed and ready to receive the patient.

CAUTION:

- STAT-BLOC™ disposable linen should always be placed in direct contact with the mattress
- STAT-BLOC™ table sheets, including premium table sheets, are not intended to be used to keep a patient in place or prevent sliding in extreme positions
- Proper positioning devices should be used per your facility protocol. Examples include but not limited to safety straps, padded footboard, bean bags, gel pads and other assistive devices¹
- AORN guidelines recommend implementing measures to prevent patients from sliding¹
- If using a patient warming device, contact the manufacturer regarding proper patient positioning and placement of the device directly under or on top of the STAT-BLOC™ disposable linen surface

REFERENCE

1. 2024 AORN Guidelines for Perioperative Practice. Positioning the Patient Guideline. AORN, Inc. 2024

ABOUT ANSELL

Ansell is a global leader in safety solutions and an integrated manufacturer of personal protection equipment for healthcare and industrial workplaces. Each day, over 10 million workers in more than 100 countries trust their safety to us. By helping workers and organizations everywhere stay two steps ahead of challenges — from workplace safety to sustainable work practices — **ANSELL IS LEADING THE WORLD TO A SAFER FUTURE.**

➔ Visit www.ansell.com/stat-bloc for more information.

* Data on file

Ansell, ® and ™ are trademarks owned by Ansell Limited or one of its affiliates. US Patented and US and non-US Patents Pending: www.ansell.com/patentmarking
© 2024 Ansell Limited. All Rights Reserved.

Lit. #130497 Rev. A

SANDEL®