


# Directions For Use

## Z-Slider® Single Pack Point-Of-Use Wall Holder

Reorder No. 2101-H

1. Place the Holder up against the wall where desired.
2. Mark the center of the three mounting holes on the wall, then remove the Holder.
3. At the mounting hole marks, drill three 1 inch deep holes using a 3/16" drill bit.
4. After the holes are drilled, use a hammer to insert the plastic anchors into the wall.
5. Place the holder into position aligned with the anchors, then screw the flathead screws until snug.

WARNING: Do not over tighten screws.

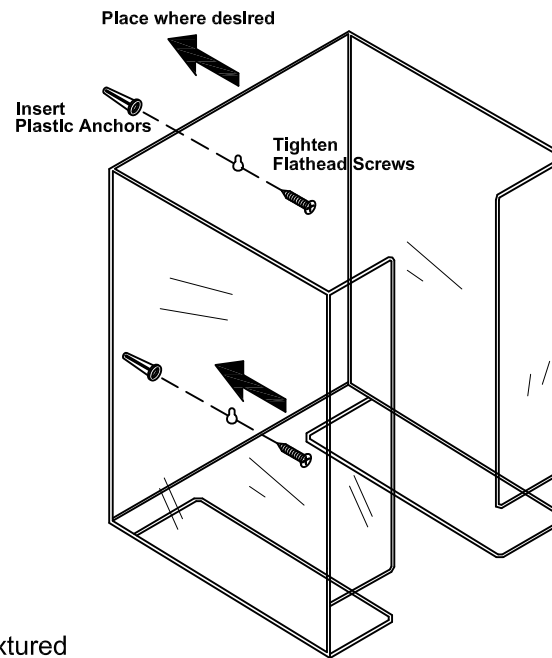

NOTE: For Low Surface Energy walls or porous textured walls Ansell Sandel Medical Solutions recommends using Screws and Anchors.

# SANDEL®

Ansell Sandel Medical Solutions LLC  
9301 Oakdale Avenue, Suite 300  
Chatsworth, CA 91311 USA  
1-818-534-2500 [www.sandelmedical.com](http://www.sandelmedical.com)