SANDEL®

Z-Slider™

REF

2101

2102

2103

2104

2151

EN

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INSTRUCTIONS FOR USE

REPOSITIONING

Step 1: The Z-Slider is designed to be placed between the draw sheet and the bed sheet.

Two caregivers should be used when repositioning a patient. With one person on each side of the bed, and using the draw sheet, log roll the patient onto their side. Place half of the Z-Slider next to the patient between the draw sheet and the bed sheet, with the arrows pointing in the direction of the move.

Log roll the patient in the opposite direction, pull the Z-Slider through the remainder of the way.

Step 2: Straighten out the Z-Slider so that it is under the patient’s body (positioned from just below the hips to the shoulders).

Step 3: While on each side of the bed, each caregiver should grasp the draw sheet and simultaneously slide the patient up in the bed to the desired position.

Step 4: The Z-Slider should be removed after repositioning the patient. Log roll the patient and remove the Z-Slider from under the patient.

LATERAL TRANSFER

Step 1: The Z-Slider should be placed between the draw sheet and the bed sheet.

Minimum of two caregivers should be used when transferring a patient. With one person on each side of the bed, and using the draw sheet, log roll the patient onto their side. Place half of the Z-slider next to the patient between the draw sheet and the bed sheet, with the arrows pointing in the direction of the move.

Log roll the patient in the opposite direction, pull the Z-Slider through the remainder of the way.

Step 2: Straighten out the Z-Slider so that it is under the patient’s body (positioned from just below the hips to the shoulders).

Step 3: While on each side of the beds and using the draw sheet, one caregiver gently pulls the draw sheet while the other caregiver gently pushes, sliding the patient from gurney to bed or bed to gurney. Care should be taken to ensure proper positioning of the head and lower legs during the transfer.

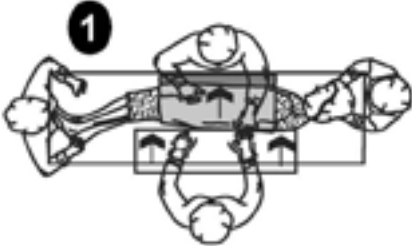
Arrows indicate the direction of the move.

Step 4: The Z-Slider should be removed after transferring the patient. Log roll the patient and remove the Z-Slider from under the patient.


Caution:

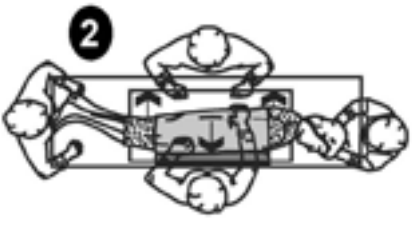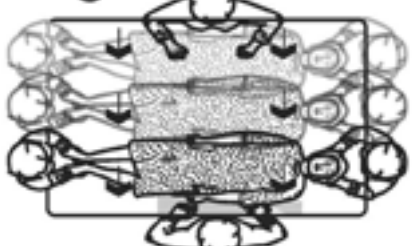
- Unless actively transferring or repositioning, do NOT leave the Z-Slider under a patient.
- Single patient use only.

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Ansell, ® and ™ are trademarks owned by Ansell Limited or one of its affiliates.
© 2020 Ansell Limited. All rights reserved.
Ansell Sandel Medical Solutions LLC
Ansell Ltd