

Lightweight, sterile antistatic coverall with hood, for protection from varied chemicals

- Reduced contamination risks: The BioClean-D Coverall with Hood - Sterile S-BDCHT features a front zip with a sealable cover and a protective flap to reduce leakage through the seams, and is manufactured in compliance with ISO cleanroom standards
- Specialized protection: This disposable coverall is manufactured using Ansell's innovative CleanTough[™] material to deliver a low-linting, antistatic, sterile coverall that protects against a range of chemicals*
- Enhanced fit: This BioClean[™] coverall is also equipped with an elasticated hood, back, cuffs and ankles, as well as thumb loops to ensure a secure hold and enhanced fit

*Please contact Ansell Customer Service for specific chemotherapy drug permeation times and recommendations.

Key Features and Benefits

- Protective flap and zip with sealable cover: Fewer contamination risks
- **CleanTough™ fabric:** For a lightweight, low-linting disposable coverall
- **Thumb loops:** Secure hold and improved fit

Industries

- Controlled and Critical Environments
- Production and Manufacturing
- Pharmaceutical Manufacturing
- Biotechnology Manufacturing
- Medical Device Manufacturing

WWW.ANSELL.COM









BioClean-D[™] Coverall with Hood - Sterile S-BDCHT

TECHNICAL DATA SHEET

PRODUCT INFORMATION

Material	CleanTough™
Audit Standards	ISO 14001, Manufacturing QMS Audit Standards ISO 9001, PPE Regulation 2016 425 Module D, NEBB Certified Cleanrooms
Standards	ASTM F739, CE 0598, EN ISO 13688:2013, EN 1149-5:2018, EN 13934-1, EN 13935-2, EN 530, EN 6530, EN 7854, EN 863, EN 9073-4, EN ISO 14325, ISO 11137-1:2006, Category III, EN ISO 13982-1:2004 + A1:2010, EN 13034:2005 + A1:2009
Packaging Overview	One piece aseptically folded per sealed inner PE bag; one inner bag per sealed outer PE bag; 20 outer bags per lined carton (20 pieces). Please note: Size 3XL and 4XL are 15 pieces per carton and subject to Minimum Order Quantity and lead time.
Country Of Origin	China
Sterilization Method	GAMMA irradiation (25 kGy)
Sterilization Minimum Dose	25kGy
Sterility Assurance Level	10-6
Cleanroom Class	Class 10/ISO Class 4 & EU GMP Grade A/B and other sterile cleanrooms
Shelf Life	Three (3) years from date of manufacture.
Construction	Bound seams with single needle stitching
Characteristics	*NOTE: BioClean CleanTough material is static dissipative and, with a charge half decay time of 0.07 sec, and so are ideal for use in a static-safe environment.

PARTICLE SHEDDING TEST RESULTS

TEST

Particle Shedding (Helmke Drum Test)

RESULT ≥ 0.5µm (counts/min) <2000

ASTM F739-12 TEST METHOD RESULTS

DRUG	Mean Breakthrough Time (MBT), Minutes Breakthrough of the test chemical is deemed to have occurred when the permeation rate has reached 0.1 µg/cm²/min
CISPLATIN	>240
CARMUSTINE	<6
CYCLOPHOSHAMIDE	217 (275,162,215)
DOXORUBICINHYDROCHLORIDE	>240
5-FLUOROURACIL	>240
METHOTREXATE	>240
ETOPOSIDE	>240
PACLITAXEL	<10
ΤΗΙΟΤΕΡΑ	30 (28,30,33)

Results achieved under controlled laboratory conditions, by accredited external testing laboratory.

SIZE CHART

S-BDCHT-XS; Size: XS, Chest: 76-84cm (30"-33"), Height: 158-164cm (5'2"-5'4") S-BDCHT-S; Size: S, Chest: 84-92cm (33"-36"), Height: 164-170cm (5'4"-5'6") S-BDCHT-M; Size: M, Chest: 92-100cm (36"-39"), Height 170-176cm (5'6"-5'9") S-BDCHT-L; Size: L, Chest: 100-108cm (39"-42"), Height: 176-182cm (5'9"-6'0") S-BDCHT-XL; Size: XL, Chest: 108-116cm (42"-45"), Height: 182-188cm (6'0"-6'2") S-BDCHT-2XL; Size: 2XL, Chest: 116-124cm (45"-48"), Height: 188-194cm (6'2"-6'4") S-BDCHT-3XL; Size: 3XL, Chest: 124-132cm (48"-51"), Height: 194-200cm (6'4"-6'6")





BioClean-D[™] Coverall with Hood - Sterile S-**BDCHT**

MATERIAL PERFORMANCE TEST RESULTS

TEST	RESULT	PERFORMANCE CLASS
Abrasion Resistance	10 to 100 cycles	1
Flex Cracking Resistance	5,000 to 15,000 cycles	3
Puncture Resistance	8 N	1
Trapezoidal Tear Resistance Cross Direction (CD)	CD 29.3 N	2
Trapezoidal Tear Resistance Machine Direction (MD)	MD 55.5 N	3
Tensile Strength Cross Direction (CD)	CD 48 N	1
Tensile Strength Machine Direction (MD)	MD 97 N	2
Repellence to Liquids – 30% H ₂ SO ₄	96.3%	3
Repellence to Liquids – 10% NaOH	97.6%	3
Repellence to Liquids – O-Xylene	95.7%	2
Repellence to Liquids – Butan-1-ol	96.6%	3
Penetration by Liquids – 30% H ₂ SO ₄	0%	3
Penetration by Liquids – 10% NaOH	0%	3
Penetration by Liquids – O-Xylene	0%	3
Penetration by Liquids – Butan-1-ol	0%	3
Inward leakage of aerosols of fine particles ¹	Pass	N/A
Resistance to penetration by spray (Spray Test) ¹	Pass	N/A
Seam Strength ²	70 N	2
Electrostatic Charge Half Decay Time, t50 (secs)	0.07	PASS

1. All interfaces with additional PPE were taped during testing, as required by the standards.

2. Seam not destroyed

ORDERING INFORMATION

	SIZE	XS, S, M, L, XL, XXL	XXXL, XXXXL
S-BDCHT	REORDER NO.	S-BDCHT-XS, S-BDCHT-S, S- BDCHT-M, S-BDCHT-L, S- BDCHT-XL, S-BDCHT-XXL	S-BDCHT-XXXL, S-BDCHT- XXXXL

Australia

UΚ

Ansell Limited

Ansell Nitritex

T: +61 1800 337 041

F: +61 1800 803 578

T: +44 1638 663338

F: +44 1638 668890

Performance Standards and Regulatory Compliance







For additional information visit us at www.ansell.com, or call us at Europe, Middle East & Africa Region

Ansell Healthcare Europe NV T: +32 (0) 2 528 74 00 F: +32 (0) 2 528 74 01

Asia Pacific Region Ansell Global Trading Center T: +603 8310 6688 F: +603 8310 6699

North America Region Ansell Healthcare Products LLC US T: +1 800 800 0444 US F: +1 800 800 0445 CA T: +1-800-363-8340

Latin America & Caribbean Region Ansell Commercial Mexico S.A. de C.V. T: +52 442 248 1544 / 248 3133

Ansell, $^{\textcircled{B}}$ and $^{\textcircled{M}}$ are trademarks owned by Ansell Limited or one of its affiliates. US Patented and US and non-US Patents Pending: www.ansell.com/patentmarking @ 2024 Ansell Limited. All Rights Reserved.

Neither this document nor any other statement made herein by or on behalf of Ansell should be construed as a warranty of merchantability or that any Ansell product is fit for a particular purpose. Ansell assumes no responsibility for the suitability or adequacy of an end user's selection of gloves for a specific application.

Please see product validation pack or contact Ansell customer service for specific data on use of garments with cytotoxic drugs. Garments used for protection against such drugs must be selected specifically for the type of chemicals used.

