

## BioClean-C<sup>™</sup> Sleeve Covers BCSC

# Non-sterile cleanroom sleeve covers, for protection against chemotherapy drugs

- Optimized protection: BioClean™
   chemotherapy-protective sleeve covers
   are specially developed to protect
   against a range of chemotherapy drugs,
   with ultrasonically bonded seams and
   protective tape, for added security
- Guaranteed cleanliness: These chemotherapy-protective garments are constructed from CleanTough™ material that is both lightweight and low-linting
- Assured barrier protection: BioClean-C<sup>™</sup> chemotherapy-protective sleeve covers have been tested against ASTM F739-12 permeation standards
- Maximized practicality: These one-sizefits-all sterile sleeve covers are tailored for quick and simple donning

### **Key Features and Benefits**

- Ultrasonically bonded seam: For sleeve covers that offer a secure fit
- Lightweight, low-linting CleanTough™ fabric: Comfort and durability
- ASTM F739-12 compliance: Assured permeation protection

### **Industries**

- Controlled and Critical Environments
- Production and Manufacturing
- Laboratory and Research









# BioClean-C<sup>™</sup> Sleeve Covers BCSC

### **TECHNICAL DATA SHEET**

### **PRODUCT INFORMATION**

| Material          | CleanTough™   |
|-------------------|---|
| Audit Standards   | Manufacturing QMS Audit Standards ISO 9001, PPE Regulation 2016 425 Module D  |
| Standards         | ASTM F739, CE 0598, EN 13034:2005 + A1:2009, EN ISO 13688:2013, EN 13935-2, EN ISO 14325, Partial Body Protection Only, Category III, EN 14605:2005 + A1:2009 |
| Country Of Origin | China   |
| Cleanroom Class   | Class 10/ISO 4  |
| Shelf Life        | Five (5) years from date of manufacture.  |
| Construction      | Ultrasonically bonded seams covered with protective tape  |
| Characteristics   | Low particulating   |

### **PARTICLE SHEDDING TEST RESULTS**

| TEST                                 | RESULT                     |
|--------------------------------------|----------------------------|
| Particle Shedding (Helmke Drum Test) | ≥ 0.5Qm (counts/min) <1700 |

### **ASTM F739-12 TEST METHOD RESULTS**

| DRUG                     | Mean Breakthrough Time (MBT), Minutes<br>Breakthrough of the test chemical is deemed to have occurred when the permeation<br>rate has reached 0.1 Qg/cm2 /min |
|--------------------------|---|
| CISPLATIN                | >240  |
| CARMUSTINE               | >240  |
| CYCLOPHOSHAMIDE          | >240  |
| DOXORUBICINHYDROCHLORIDE | >240  |
| 5-FLUOROURACIL           | >240  |
| METHOTREXATE             | >240  |
| ETOPOSIDE                | <6  |
| PACLITAXEL               | >240  |
| THIOTEPA                 | 55 (46,88,31)   |

Results achieved under controlled laboratory conditions, by accredited external testing laboratory.

### **SIZE CHART**

BCSC; Size: Universal





## **BioClean-C™ Sleeve Covers BCSC**

### MATERIAL PERFORMANCE TEST RESULTS

| TEST  | RESULT           | PERFORMANCE CLASS |
|---|------------------|-------------------|
| Abrasion Resistance   | 10 to 100 cycles | 1                 |
| Puncture Resistance   | 10 N             | 1                 |
| Trapezoidal Tear Resistance Cross Direction (CD)            | CD 32.3 N        | 2                 |
| Trapezoidal Tear Resistance Machine Direction (MD)          | MD 50.6 N        | 3                 |
| Tensile Strength Cross Direction (CD)                       | CD 47 N          | 1                 |
| Tensile Strength Machine Direction (MD)                     | MD 94 N          | 2                 |
| Repellence to Liquids - 30% H <sub>2</sub> SO <sub>4</sub>  | 99.0%            | 3                 |
| Repellence to Liquids - 10% NaOH                            | 98.4%            | 3                 |
| Repellence to Liquids - O-Xylene                            | 91.4%            | 1                 |
| Repellence to Liquids - Butan-1-ol                          | 93.9%            | 3                 |
| Penetration by Liquids - 30% H <sub>2</sub> SO <sub>4</sub> | 0%               | 3                 |
| Penetration by Liquids - 10% NaOH                           | 0%               | 3                 |
| Penetration by Liquids - O-Xylene                           | 0%               | 3                 |
| Penetration by Liquids - Butan-1-ol                         | 0%               | 3                 |
| Seam Strength <sup>1</sup>                                  | 74 N             | 2                 |

<sup>1.</sup> Seam not destroyed

### **ORDERING INFORMATION**

|      | SIZE        | BioClean-C Chemo Sleeve Covers BCSC |
|------|-------------|-------------------------------------|
| BCSC | REORDER NO. | BCSC                                |

### For additional information visit us at www.ansell.com, or call us at

### Europe, Middle East & Africa Region North America Region

Ansell Healthcare Europe NV T: +32 (0) 2 528 74 00 F: +32 (0) 2 528 74 01

### Asia Pacific Region

Ansell Global Trading Center T: +603 8310 6688 F: +603 8310 6699

Ansell Healthcare Products LLC US T: +1 800 800 0444 US F: +1 800 800 0445 CA T: +1-800-363-8340

#### Latin America & Caribbean Region

 Latin America & Caribbean Region
 Ansell Nitritex

 Ansell Commercial Mexico S.A. de C.V.
 T: +44 1638 663338

 T: +52 442 248 1544 / 248 3133
 F: +44 1638 668890

### Australia

Ansell Limited T: +61 1800 337 041 F: +61 1800 803 578

**Performance Standards and Regulatory Compliance** 









Ansell,  $^{\textcircled{R}}$  and  $^{\textcircled{M}}$  are trademarks owned by Ansell Limited or one of its affiliates. US Patented and US and non-US Patents Pending: www.ansell.com/patentmarking  $^{\textcircled{M}}$  2024 Ansell Limited. All Rights Reserved.

Neither this document nor any other statement made herein by or on behalf of Ansell should be construed as a warranty of merchantability or that any Ansell product is fit for a particular purpose. Ansell assumes no responsibility for the suitability or adequacy of an end user's selection of gloves for a specific application.



